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Lyasis

Rua Cromita 278 - Distrito Industrial - Itabira - MG
Telefax: 31 3834-6400
e.mail: dsa@invitro.com.br

IN VITRO DIAGNÓSTICA

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ÁCIDO ÚRICO ENZIMÁTICO

Cat.: 10689	Volume: 50 mL	Nº testes: 166
Cat.: 10687	Volume: 100 mL	Nº testes: 333
Cat.: 10688	Volume: 250 mL	Nº testes: 833

NAME Ácido Úrico

Unit	mg/dL	Unit Factot	1	Decimals	1
Type	End point	Direction	UP		
Linearity		Low	0	High	20
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	1
Filter1	540	Filter2	No	Bich. factor	
Automatic Rerun		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	3	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1 Volume	300	Delay/Inc.	06	reading	Rinse	30	Cod. rgt	Acu
RGT2								

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Nº testes: 1666

NAME ALBUMINA

Unit	g/dL	Unit Factot		Decimals	1
Type	End point	Direction	UP		
Linearity		Low	0	High	6.0
On Board		RBL	7	Calibration	7
Stability		Mode1	Std/fator	Factor	1
Standard	Yes	Filter2	No	Bich. factor	
Filter1	600	Without Diluition	No	With Diluition	Yes
Automatic Rerum		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	3	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1 Volume	300	Delay/Inc.	60	reading	Rinse	30	Cod. rgt	ALB
RGT2								

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Nº testes: 480

NAME Alfa Amilase

Unit Type	U/L kinetic	Unit Factot Direction	1 UP	Decimals	0
Linearity		Low	0	High	7000
On Board Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	24820
Filter1	405	Filter2		Bich. factor	
Automatic Rerum		Without Diluition	Yes	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.500	High	1.200
Correlation		Slope	0.9	Intercept	0
Sample Vol.	3	Sample Blank	No		
E.P. limit	1.8	Fit	0.998	Rgt Rate	No

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Nº testes: 180

NAME Bilirrubina Direta

Unit	mg/dL	Unit Factot	1	Decimals	0
Type	End point	Direction	UP		
Linearity		Low	0	High	30
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	13.0
Filter1	570	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	No	With Diluition	No
		Dil./ Ratio	2		
Stability		Low	0.00	High	0.05
Correlation		Slope	1	Intercept	0
Sample Vol.	20	Sample Blank	Yes		
E.P. limit	0.005	Fit		Rgt Rate	

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Nº testes: 666

NAME Cálcio

Unit	mg/dL	Unit Factot	1	Decimals	0
Type	End point	Direction	UP		
Linearity		Low	0	High	25
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	
Filter1	570	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.80
Correlation		Slope	1	Intercept	0
Sample Vol.	8	Sample Blank	No		
E.P. limit	0.005	Fit		Rgt Rate	

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Nº testes: 333

NAME CK - MB

Unit Type	U/L kinetic	Unit Factot Direction	1 UP	Decimals	0
Linearity		Low	0	High	2000
On Board Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	8500
Filter1	340	Filter2		Bich. factor	
Automatic Rerum		Without Diluition	Yes	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.000	High	1.400
Correlation		Slope	0.9	Intercept	0
Sample Vol.	15	Sample Blank	No		
E.P. limit	1.8	Fit	0.998	Rgt Rate	No

RGT1 Volume **300** Delay/Inc. **180** reading **60** Rinse Cod. rgt **CKMB**
RGT2

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Cat.: 12015 Volume: 100 mL Nº testes: 333

NAME CK NAC

Unit	U/L	Unit Factot	1	Decimals	0
Type	kinetic	Direction	UP		
Linearity		Low	0	High	1000
On Board		RBL	7	Calibration	7
Stability		Mode1	Std/fator	Factor	8254
Standard	Yes	Filter2		Bich. factor	
Filter1	340	Without Diluition	Yes	With Diluition	Yes
Automatic Rerum		Dil./ Ratio	2		
Stability		Low	0.500	High	1.200
Correlation		Slope	0.9	Intercept	0
Sample Vol.	8	Sample Blank	No		
E.P. limit	1.8	Fit	0.998	Rgt Rate	No

RGT1 Volume	300	Delay/Inc.	180	reading	Rinse	54	Cod. rgt	CKNAC
RGT2								

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Nº testes: 1666

NAME Colesterol

Unit	mg/dL	Unit Factot	1	Decimals	0
Type	End point	Direction	UP		
Linearity		Low	0	High	750
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	1
Filter1	510	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	3	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1 Volume **300** Delay/Inc. **306** reading Rinse **30** Cod. rgt **CoIT**
RGT2

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Cat.: 044 Volume: 25 mL Nº testes: 62

NAME Colesterol HDL
precipitante

Unit	mg/dL	Unit Factot	1	Decimals	0
Type	End point	Direction	UP		
Linearity		Low	0	High	750
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	1
Filter1	510	Filter2	No	Bich. factor	
Automatic Rerun		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	20	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1 Volume	300	Delay/Inc.	306	reading	Rinse30	Cod. rgt	HDLp
RGT2							

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Nº testes: 200

NAME HDL Direto

Unit	mg/dL	Unit Factot	1	Decimals	0
Type	End point	Direction	UP		
Linearity		Low	0	High	200
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	1
Filter1	600	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	4	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1	Volume	300	Delay/Inc.	240	reading	Rinse	Cod. rgt	HDL1
RGT2		100		240				HDL2

Last Calibration Results				
Flag	3	Concentration	OD *	Recalcul. Conc.
%Error		Absolute Error		

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Nº testes: 200

NAME LDL Direto

Unit	mg/dL	Unit Factot	1	Decimals	0
Type	End point	Direction	UP		
Linearity		Low	0	High	1000
On Board		RBL	7	Calibration	7
Stability	Yes	Mode1	Std/fator	Factor	1
Standard	600	Filter2	No	Bich. factor	
Filter1		Without Diluition	No	With Diluition	Yes
Automatic Rerum		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	4	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1 Volume	300	Delay/Inc.	300	reading	Rinse	Cod. rgt	LDL1
RGT2	100		240				LDL2

Last Calibration Results			
Flag	3	Concentration *	OD
%Error		Absolute Error	Recalcul. Conc.

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Nº testes: 4166

NAME Creatinina

Unit	mg/dL	Unit Factot	1	Decimals	1
Type	kinetic	Direction	UP		
Linearity		Low	0	High	10.0
On Board		RBL	7	Calibration	7
Stability		Mode1	Std/fator	Factor	
Standard	Yes	Filter2	No	Bich. factor	
Filter1	510	Without Diluition	No	With Diluition	Yes
Automatic Rerum		Dil./ Ratio	2		
Stability		Low	0.04	High	1.80
Correlation		Slope	0.9	Intercept	0
Sample Vol.	15	Sample Blank	No		
E.P. limit	0.300	Fit	0.998	Rgt Rate	No

RGT1 Volume **300** Delay/Inc. **60** reading **30** Rinse Cod. rgt **Cre**
RGT2

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Cat.: 12660 Volume: 32 mL Nº tetes: 106

NAME FACT

Unit	U/L	Unit Factot	1	Decimals	0
Type	Kinetic	Direction	UP		
Linearity		Low	0	High	74
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	248
Filter1	405	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	Yes	With Diluition	No
		Dil./ Ratio	2		
Stability		Low	0.500	High	1.20
Correlation		Slope	0.9	Intercept	0
Sample Vol.	20	Sample Blank	No		
E.P. limit	1.8	Fit	0.998	Rgt Rate	No

RGT1 Volume	300	Delay/Inc.	300	reading	Rinse	Cod. rgt	FACT
RGT2							

Last Calibration Results			
Flag	3	Concentration	OD
%Error		Absolute Error	Recalcul. Conc.

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Nº tetes: 106

NAME FACP

Unit Type	U/L	Unit Factot	1	Decimals	0
Linearity	Kinetic	Direction	UP	High	74
On Board		Low	0	Calibration	7
Stability		RBL	7	Factor	248
Standard	Yes	Mode1	Std/fator	Bich. factor	
Filter1	405	Filter2	No	With Diluition	No
Automatic Rerum		Without Diluition	Yes	High	1.20
Stability		Dil./ Ratio	2	Intercept	0
Correlation		Low	0.500	Rgt Rate	No
Sample Vol.	20	Slope	0.9		
E.P. limit	1.8	Sample Blank	No		
		Fit	0.998		

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Nº testes: 3333

NAME FAL

Unit Type	U/L kinetic	Unit Factot Direction	1 UP	Decimals	0
Linearity		Low	10	High	1400
On Board Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	2179
Filter1	405	Filter2		Bich. factor	
Automatic Rerum		Without Diluition	Yes	With Diluition	No
		Dil./ Ratio	2		
Stability		Low	0.500	High	1.200
Correlation		Slope	0.9	Intercept	0
Sample Vol.	6	Sample Blank	No		
E.P. limit	1.8	Fit	0.998	Rgt Rate	No

RGT1 Volume **300** Delay/Inc. **162** reading **54** Rinse Cod. rgt **Fal**
RGT2

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Nº testes: 666

NAME Fósforo

Unit	mg/dL	Unit Factot	1	Decimals	1
Type	End point	Direction	UP		
Linearity		Low	0	High	30
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	1
Filter1	340	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	5	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1 Volume **300** Delay/Inc. **180** reading Rinse **30** Cod. rgt **Fosf**
RGT2

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Nº testes: 4000

NAME GGT

Unit	U/L	Unit Factot	1	Decimals	0
Type	kinetic	Direction	UP		
Linearity		Low	0	High	230
On Board		RBL	7	Calibration	7
Stability	Yes	Mode1	Std/fator	Factor	1158
Standard	405	Filter2		Bich. factor	
Filter1		Without Diluition	Yes	With Diluition	No
Automatic Rerum		Dil./ Ratio	2		
Stability		Low	0.500	High	1.200
Correlation		Slope	0.9	Intercept	0
Sample Vol.	25	Sample Blank	No		
E.P. limit	1.8	Fit	0.998	Rgt Rate	No

RGT1 Volume **250** Delay/Inc. **60** reading **54** Rinse Cod. rgt **GGT**
RGT2

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Nº testes: 3333

NAME Colesterol

Unit	mg/dL	Unit Factot	1	Decimals	0
Type	End point	Direction	UP		
Linearity		Low	0	High	400
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	1
Filter1	510	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	3	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

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NAME GOT

Unit	U/L	Unit Factot	1	Decimals	0
Type	kinetic	Direction	UP		
Linearity		Low	0	High	350
On Board		RBL	7	Calibration	7
Stability	Yes	Mode1	Std/fator	Factor	1745
Standard	340	Filter2		Bich. factor	
Filter1		Without Diluition	Yes	With Diluition	No
Automatic Rerum		Dil./ Ratio	2		
Stability		Low	0.500	High	1.200
Correlation		Slope	0.9	Intercept	0
Sample Vol.	25	Sample Blank	No		
E.P. limit	1.8	Fit	0.998	Rgt Rate	No

RGT1 Volume **250** Delay/Inc. **60** reading **54** Rinse Cod. rgt **GOT**
 RGT2

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NAME GPT

Unit	U/L	Unit Factot	1	Decimals	0
Type	kinetic	Direction	UP		
Linearity		Low	0	High	600
On Board		RBL	7	Calibration	7
Stability		Mode1	Std/fator	Factor	1745
Standard	Yes	Filter2		Bich. factor	
Filter1	340	Without Diluition	Yes	With Diluition	No
Automatic Rerum		Dil./ Ratio	2		
Stability		Low	0.500	High	1.200
Correlation		Slope	0.9	Intercept	0
Sample Vol.	25	Sample Blank	No		
E.P. limit	1.8	Fit	0.998	Rgt Rate	No

RGT1 Volume **250** Delay/Inc. **60** reading **54** Rinse Cod. rgt **GPT**
 RGT2

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Nº testes: 1333

NAME LDH

Unit	U/L	Unit Factot	1	Decimals	0
Type	kinetic	Direction	UP		
Linearity		Low	0	High	1300
On Board					7
Stability		RBL	7	Calibration	20000
Standard	Yes	Mode1	Std/fator	Factor	
Filter1	340	Filter2		Bich. factor	No
Automatic Rerum		Without Diluition	Yes	With Diluition	
		Dil./ Ratio	2		1.200
Stability		Low	0.500	High	0
Correlation		Slope	0.9	Intercept	
Sample Vol.	3	Sample Blank	No		
E.P. limit	1.8	Fit	0.998	Rgt Rate	No

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Nº testes: 666

NAME MAG

Unit	mg/dL	Unit Factot		Decimals	1
Type	End point	Direction	UP		
Linearity		Low	0	High	5
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	1
Filter1	520	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	3	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1 Volume **300** Delay/Inc. **240**reading Rinse **30** Cod. rgt **MAG**
RGT2

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Nº testes: 1666

NAME PT

Unit	g/dL	Unit Factot		Decimals	1
Type	End point	Direction	UP		
Linearity		Low	0	High	12
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	1
Filter1	600	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	6	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1 Volume **300** Delay/Inc. **240** reading Rinse **30** Cod. rgt **PT**
RGT2

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NAME TRI

Unit	mg/dL	Unit Factot	1	Decimals	0
Type	End point	Direction	UP		
Linearity		Low	0	High	1000
On Board					
Stability		RBL	7	Calibration	7
Standard	Yes	Mode1	Std/fator	Factor	1
Filter1	510	Filter2	No	Bich. factor	
Automatic Rerum		Without Diluition	No	With Diluition	Yes
		Dil./ Ratio	2		
Stability		Low	0.01	High	1.15
Correlation		Slope	1	Intercept	0
Sample Vol.	3	Sample Blank	No		
E.P. limit	0.05	Fit		Rgt Rate	

RGT1 Volume **300** Delay/Inc. **306** reading Rinse **30** Cod. rgt **TRI**
 RGT2

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Nº testes: 1000

NAME URÉIA UV

Unit	mg/dL	Unit Factot	1	Decimals	0
Type	Cinética	Direction	UP		
Linearity		Low	0	High	300
On Board		RBL	7	Calibration	7
Stability		Mode1	Std/fator	Factor	
Standard	Yes	Filter2	No	Bich. factor	
Filter1	340	Without Diluition	Yes	With Diluition	No
Automatic Rerum		Dil./ Ratio	2		
Stability		Low	0.5	High	1.2
Correlation		Slope	0,9	Intercept	0
Sample Vol.	3	Sample Blank	No		
E.P. limit	1,80	Fit	0,998	Rgt Rate	No

RGT1 Volume **300** Delay/Inc. **60** reading **54** Rinse **30** Cod. rgt **URUV**
RGT2

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